International Instructions for Use

Xlevator

Languages:

EN, DE, SL

fEX Engineering GmbH

Höftenweg 49
48155 Münster
Germany

Third edition

Publication date 22/07/2024

Foreword

Thank you for purchasing an **Xlevator** chair bracket from fEX Engineering GmbH.

The Xlevator guarantees that the powered **Xpedition** stair chair made by Stryker is properly secured. Power-assisted linear movement ensures ergonomic handling when removing or stowing it in a rescue vehicle.

WARNING

Improper handling or any unintended use may cause incur risk or cause damage. Please therefore read and carefully follow these Instructions for use. Retain them for reference at all times. To prevent personal injury or damage to property, please observe all safety precautions.

To contact the manufacturer:

fEX Engineering GmbH
Höltenweg 49
48155 Münster
Germany
engineering@fex.group
+49 (0)251 508 537 -13

Contents

1	How to use these Instructions for use.....	1
1.1	Feedback on these Instructions for use	1
1.2	Explanation of the warning notices used in these Instructions for use	1
2	Intended purpose	2
2.1	Intended use.....	2
2.2	Intended purpose	2
2.3	Intended use with other products.....	2
2.4	Intended users.....	2
2.5	Intended environment for use	2
3	Important safety notice.....	3
3.1	Reporting incidents	3
4	Product description	4
4.1	How it works.....	4
4.2	Symbols used.....	4
4.3	Product overview	5
4.4	Use in combination with other products	6
4.5	Delivery.....	6
4.6	Modifications.....	6
4.7	Requirements for storage, handling and environmental conditions	6
4.8	Product service life	6
4.9	Warranty.....	6
5	Using the product.....	7
5.1	Operator’s duty of care	7
5.2	User’s duty of care.....	7
5.3	Instructions for safe operation.....	7
5.4	Introduction.....	7
5.5	Adjusting the height and locking.....	8
5.6	Empty trip	8
5.7	Removal.....	9
5.8	Stowing.....	9
5.9	Emergency mode.....	10

6	Reprocessing: Cleaning and disinfection	11
6.1	Cleaning the product	11
6.2	Disinfectants	11
6.3	Disinfecting the product	12
6.4	Final treatment	12
7	Maintenance	13
7.1	Maintenance	13
7.2	Visual inspections	14
7.3	Troubleshooting	15
7.4	Maintenance by users	16
7.5	Changing the bar clasp	16
7.6	Replacing the handle	17
7.7	Replacing a strap	17
7.8	Replacing the strap holder	18
7.9	Replacing the wheel mount(s)	18
7.10	Replacing the edge protector	19
7.11	Replacing safety precautions and reflective strips	19
7.12	Replacing the holding plates	20
8	Dismantling, recycling, disposal	21
8.1	Dismantling	21
8.2	Recycling, disposal	21
8.3	Disposal through fEX Engineering GmbH	21
9	Risk management	22
9.1	Foreseeable misuse	22
9.2	Residual risk	22
10	List of individual parts	23

1 How to use these Instructions for use

If you have any questions about the contents of these Instructions for use or how to use the product, please contact us. You can request a paper copy of these instructions for use or download the current, electronic version from our website:

<https://www.fex.group/downloads>

Should you need further assistance after carefully reading these Instructions for use and additional information, please contact your authorised distributor or contact us directly.

1.1 Feedback on these Instructions for use

Your opinion is important to us. Please let us know if there is anything you would change about these instructions for use (engineering@fex.group). We will take your feedback into account in the next version of these Instructions for use.

1.2 Explanation of the warning notices used in these Instructions for use

Signal word	Description
 WARNING	Potentially hazardous situation resulting in severe injury or death.
 CAUTION	Potential hazard resulting in minor or moderate injury.
NOTE	User error can damage the product.

Table 1: Explanation of the warning notices used in these Instructions for use

2 Intended purpose

2.1 Intended use

The Xlevator is designed as a professional securing device for the powered **Xpedition** stair chair made by *Stryker*. The system is intended to raise and lower the *Xpedition Chair* between its stowed position and ground level. Power-assisted linear movement ensures ergonomic handling when removing or stowing it in a rescue vehicle.

2.2 Intended purpose

The Xlevator is a bracket and lifting aid for the *Xpedition Chair* made by *Stryker*.

2.3 Intended use with other products

The Xlevator is intended exclusively for use with the *Xpedition* made by *Stryker*. It is designed for permanent installation in rescue vehicles. The screw connections in the vehicle must be bale to withstand a force of 5000 N.

2.4 Intended users

The Xlevator is intended for use in a professional healthcare environment. The users, and the qualifications they require, are specified accordingly.

User groups	Emergency services personnel (primary users)	Medical technicians, technical managers, cleaning staff, installation technicians (secondary users)
Training, certification	Medical training, e.g. as an emergency paramedic, emergency medical technician, rescuers, emergency services support staff, and so on.	Technical training, medical technician training or equivalent, or accreditation as a cleaner in a professional healthcare environment, or training as an installation technician

Table 2: Intended users

2.5 Intended environment for use

The Xlevator is intended for use by primary users in the following environments:

- Emergency services vehicles
- Mobile environments

3 Important safety notice

NOTE

Please read and observe the instructions in these Instructions for use to use this system safely.

⚠ WARNING

The *Xpediton* must be properly stowed and fully strapped in. Otherwise there is a risk of the *Xpediton* detaching from the Xlevator and becoming a projectile in the event of a traffic accident.

3.1 Reporting incidents

We want to continually improve the product and minimise potential risks. Please report any incidents (damage, injuries, etc.) related to the product to the manufacturer, distributor, authorised representative or importer.

To contact the manufacturer:

fEX Engineering GmbH
Höltenweg 49
48155 Münster
Germany
+49 (0)251 508 537 -13
engineering@fex.group

4 Product description

4.1 How it works

The Xlevator is an accessory for the powered *Xpedition* stair chair made by *Stryker*. It consists of a frame, a wall plate and movable carriage and a chair plate. These two elements are connected to one another with a linear movement, and can therefore be moved up and down. A gas spring and an oil brake are installed between the chair plate and the wall plate. The gas spring supports upward movement to help the user to stow the *Xpedition* in the emergency services vehicle with ease. In contrast, the oil brake ensures that the chair plate rises in a controlled manner on empty trips. The built-in locking mechanism enables the *Xpedition* to be secured in incremental vertical positions. The handle bar of the *Xpedition* locks behind the bar clasp at the top of the chair plate. The wheels fit into two wheel mounts at the bottom of the chair plate. The *Xpedition* is then secured with two straps.

4.2 Symbols used

Figure 1: Labels

Figure 2: Safety notice "Caution: Crush hazard"

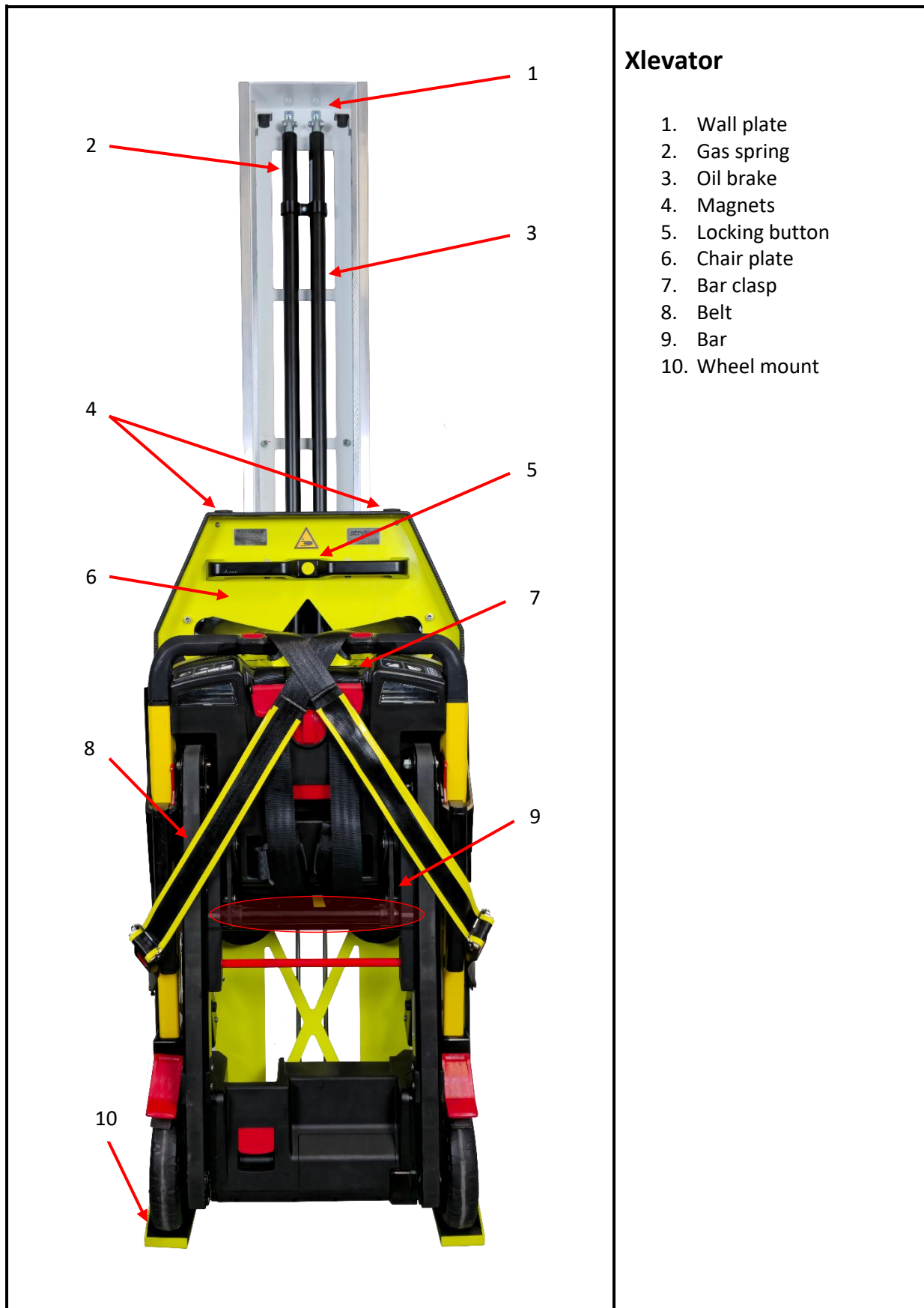
Symbol	Meaning
	Manufacturer's logo
	Manufacturer, address and contact details
	Conformity marking
	Read Instructions for use
	Caution: Crush hazard
S/N	2024 03 06 001 Year of construction/Month/Date/Batch number

Table 3: Symbols used and their meanings

4.3 Product overview

Xlevator

- 1. Wall plate
- 2. Gas spring
- 3. Oil brake
- 4. Magnets
- 5. Locking button
- 6. Chair plate
- 7. Bar clasp
- 8. Belt
- 9. Bar
- 10. Wheel mount

Table 4: Product overview

4.4 Use in combination with other products

The Xlevator is designed exclusively for carrying and securing the powered Xpedition stair chair made by *Stryker*. Only this product can be stowed and transported in adequate safety.

4.5 Delivery

The product is delivered in non-sterile, non-disinfected condition.

An electronic copy of the Instructions for use can be found via the following link:

<https://www.fex.group/downloads/>

4.6 Modifications

Unauthorised modifications to this product are prohibited.

4.7 Requirements for storage, handling and environmental conditions

Category	Information
Permitted humidity limits	5% to 98%
Permitted temperature limit (vehicle and storage)	10°C to +50°C
Maximum storage time before use. Cylinders can remain in the same position for a maximum of 6 months.	6 months

Table 5: Storage during transport, before and after use

4.8 Product service life

Under normal usage conditions and with appropriate maintenance, the service life is **seven years**.

The date for the product’s service life is based on the delivery date from fEX Engineering.

4.9 Warranty

fEX Engineering provides the statutory warranty of **two years** under EU and German law for the Xlevator.

The oil brake, gas spring and straps are excluded from the above warranty. A **one-year** warranty applies to these components.

5 Using the product

5.1 Operator's duty of care

To ensure safe operation, the operator must meet the following obligations:

- Personnel must be trained to use the Xlevator installed
- The Instructions for use must be read and followed
- Products must be assembled or installed properly
- Only suitable, qualified, trained specialist personnel may use the product
- All serious incidents (damage, injuries, etc.) must be reported to the manufacturer.

5.2 User's duty of care

To ensure safe operation, the user must meet the following obligations:

- The Instructions for use must be read and followed
- The product must be used as intended
- Damage must be reported to the technical manager

5.3 Instructions for safe operation

- Ensure product is fully locked
- Perform weekly inspections (Section 8.2 *Inspection*)
- Only use the product for its intended purpose
- Only use the product with the *Xpedition* stair chair

5.4 Introduction

The Xlevator has a handle with a locking button in the centre. The Xlevator and its locking mechanism are operated using these features. The *Xpedition* is secured to the chair plate by two straps. The strap lock is opened by the push button. Once opened, the straps can be fastened to the magnets above the chair plate to prevent obstructions when stowing. The position of these parts is illustrated in Section 4.2 *Product overview*.

NOTE

The Xlevator must be in the upper position before the vehicle or door is moved. If the system is in the lowest position, the vehicle height can only be minimally lowered. This may occur when loading/entering the vehicle, or when using air suspension. Otherwise the system may be damaged.

A video on how to use the Xlevator is available on the manufacturer's website.

5.5 Adjusting the height and locking

NOTE

The oil brake controls the speed of upward travel. Raise the system slowly to move it with ease.

CAUTION

The Xlevator must never be used for passenger transport. Never load the Xlevator with someone's full body weight or when it is locked.

1. Push the locking button.
2. The lock is now open and the height can be set.
3. Move the chair plate to the required height. Lift it using the connecting bar on the track drive (Table 4) on the *Xpedition*.
4. Continue moving the chair plate until the lock clicks into place at the next possible point.

5.6 Empty trip

If the stair chair is not in the system, its weight is missing and an empty trip to a lower position requires force. To make it easier and lighter to handle, follow the steps below:

1. Hold the handle with both hands and press the locking button.
2. Use your body weight to pull the system down until the central position locks. Do not pull on the bar clasp of the Xlevator (Fig. 3).
3. Put your foot on one of the wheel mounts.
4. Press the locking button and move the system to the lowest position using your body weight.

CAUTION

Hold the handle firmly with both hands to avoid a crush hazard.

NOTE

The gas spring continuously pulls the system upwards. Counteract this force with your body weight before opening the lock.

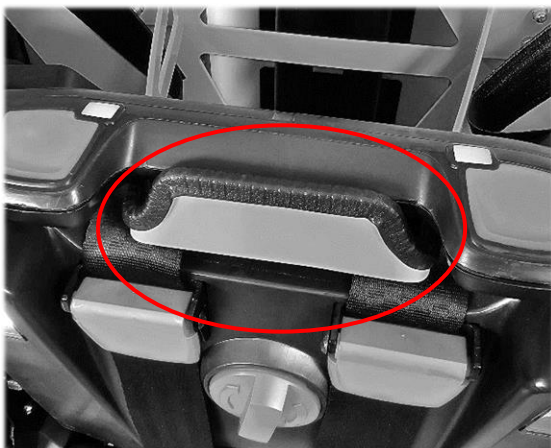
Figure 3: Bar clasp

5.7 Removal

1. Now move the chair plate to its lowest position.
2. Loosen the straps, attach the right strap to the right magnet and the left strap to the left magnet (Fig. 4).
3. Lift the handle bar of the *Xpedition* and roll the stair chair out of the Xlevator.
4. If necessary, the Xlevator can be raised to its highest position without the stair chair.

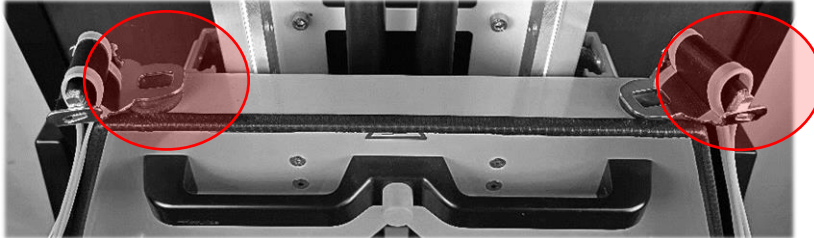
Figure 4: Position of the magnets

5.8 Stowing

WARNING

The *Xpedition* must be properly stowed and fully and tightly strapped in. If it is not, there is a risk of the *Xpedition* detaching from the Xlevator and becoming a projectile in the event of a traffic accident.

1. If necessary, move the Xlevator down to its lowest position.
2. Fold the *Xpedition* and move the wheels so they are in front of the wheel mounts. The handle bar of the *Xpedition* must remain extended.
3. First lift the wheels of the *Xpedition* into the wheel mounts of the Xlevator. To make it as easy as possible, hold the handle of the *Xpedition* with both hands and lift the *Xpedition* with your foot.
4. Both wheels must be fully seated in the wheel mounts.
5. Move the handle bar back so that it sits behind the bar clasp (Fig. 5).
6. Strap in the *Xpedition*. The straps must cross diagonally over the handle bar (Fig. 6).
7. Tighten the straps firmly at the buckles.
8. Then move the Xlevator to its highest position.

NOTE

Always make sure that the handle bar of the *Xpedition* is completely secured behind the bar clasp of the Xlevator (Fig. 5)

Figure 5: Bar clasp

Figure 6: Strap attached

5.9 Emergency mode

If any of the components fail, e.g. the locking mechanism or the linear movement, the Xlevator can be used in emergency mode to allow it to be used as a static bracket. This ensures that the vehicle can be used until repairs can be carried out. To do this, screw in an M6-x-60-mm screw from the outside of the guide rails on both sides (Fig. 7).

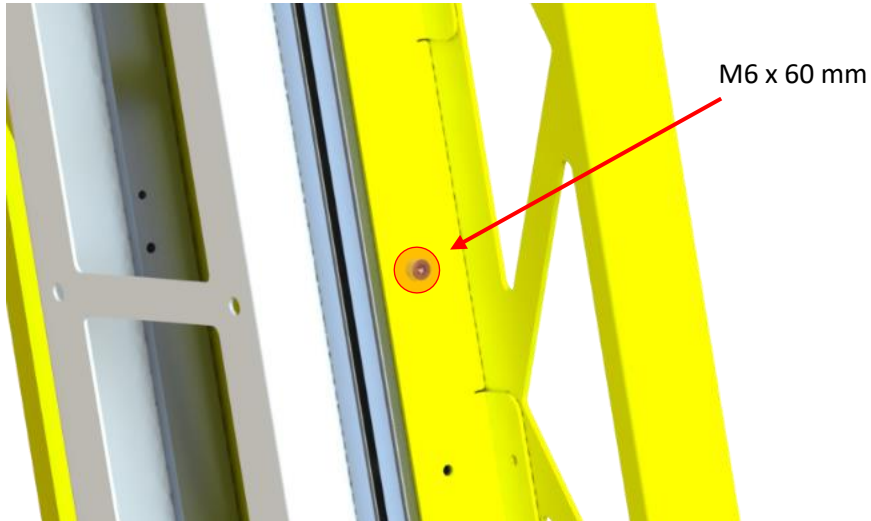
Figure 7: Screw for emergency mode

6 Reprocessing: Cleaning and disinfection

Cleaning and disinfection must follow the process in the Instructions for use, but must also be carried out after every potential contamination, however at least once a month.

We recommend cleaning the product at the same time as inspecting it. If possible, combine this with existing weekly routines (e.g. when cleaning the vehicle).

NOTE

The Xlevator must **not** be cleaned with high pressure.

6.1 Cleaning the product

1. Wipe the product's surfaces with a solution of mild soap and water.
2. Dry all product surfaces thoroughly.

6.2 Disinfectants

Agents with the following ingredients can be used for disinfection:

- Quaternary cleaners (active ingredient: ammonium chloride) containing less than 3% glycol ether
- Chlorinated bleaching agents (5.25 % – less than 1 part bleaching agent to 100 parts of water)
- Phenolic cleaners (active ingredient: o-phenylphenol)
- ≤ 21% isopropanol

Disinfectants made from quaternary ammonium compounds must be used in accordance with the recommended instructions for use. Quaternary cleaners and disinfectants must be used with extreme care on plastic surfaces.

⚠ WARNING

Disinfectants containing one of the following basic ingredients are not approved as they can damage the system:

- Hydrogen peroxide
- (Per-/Chlor-) acetic acid

6.3 Disinfecting the product

1. Remove any visible contaminants from the product (Section 7.1 *Cleaning the product*).
2. Apply the recommended disinfectant solution. Do not soak.
3. Allow the product to air dry. Follow the instructions and exposure time as directed by the manufacturer of the chemical products.

Using an unsuitable agent, an excessive amount of an agent, an exposure time that is too long or failing to neutralise an agent sufficiently may damage the equipment. All of the information provided by the manufacturer of the agents used is decisive and must always be observed.

6.4 Final treatment

Residues of cleaning agents must be removed. After disinfection, always wipe the product with a clean, water-soaked cloth and allow it to air dry.

 **WARNING**

Residues of cleaning agents are corrosive and can injure users and damage the product.

7 Maintenance

The operator and his staff can maintain the product by following the instructions. Unauthorised repairs to this product are prohibited. The operator is permitted to replace certain components; these are divided into two groups:

1. Components that can be replaced by **users (laymen)** (Section 7.5 *Maintenance by users*), e.g. the straps
2. Components that must be replaced by **technicians (instructions in the separate document: *Installation and repair instructions - Xlevator*)**, e.g. the gas spring

It is the operator's responsibility to employ suitable, qualified and trained personnel carry this out.

Medium-strength screw locking is strongly recommended. The required curing time depends on the information given by the screw locking manufacturer.

Only replacement parts made by fEX Engineering GmbH may be used. Using other parts may be dangerous. Repairs are prohibited.

The Instructions for use are decisive for all work on the product. Once damaged components have been replaced, the system can be put back into use.

fEX Engineering GmbH refuses to supply spare parts for or repair any products that have reached the end of their service life.

7.1 Maintenance

The guide rails must be lubricated as required. This may vary depending on e.g. the frequency/intensity of cleaning, but must take place **at least once a year**.

The document *Assembly and repair instructions - Xlevator* describes lubrication in detail.

Weekly visual inspections must be carried out (Sec. 7.2 *Visual inspections*).

Faulty components must be replaced as necessary.

7.2 Visual inspections

⚠ CAUTION

Damage to the system can create a safety hazard during use. Any damage must be immediately reported to the technical manager.

Functional tests and visual inspections for damage must be carried out weekly.

We recommend inspecting the product when cleaning it. If possible, combine this with existing routines (e.g. when cleaning the vehicle). If damage or impaired functionality are identified using the following list, any faults must be rectified (Section 7.3 *Troubleshooting*), or the relevant components must be replaced (Section 7.4 *Maintenance by users* and the document *Assembly and repair instructions - Xlevator*).

⚠ WARNING

There must be no visible damage on the straps, and they must be fully functional. Otherwise, there is a risk of the *Xpedition* becoming a projectile in the event of a traffic accident.

Check the Xlevator for the following functionality and possible damage:

- a) Damage to straps or locks not working
- b) Gas spring supports upward movement
- c) Oil brake slows upward movement (empty trip)
- d) Gas spring and oil brake are not leaking
- e) Unhindered linear movement (system can be moved up and down)
- f) Locking mechanism works in all positions
- g) Safety notices are legible
- h) Rust formation or other material damage (sharp edges, brittleness or cracks)
- i) Other damage to the product

This does not apply to visual defects such as superficial scratches that cannot cause injury.

⚠ WARNING

If the vehicle is withdrawn from use because of an accident, the Xlevator installed inside it must **not** be reused.

7.3 Troubleshooting

Users or technical managers can carry out troubleshooting. Use Table 6: *Troubleshooting*. If a problem cannot be solved or a repair is needed, please contact us.

Problem	Possible cause	Troubleshooting
Straps will not close	Foreign body in the lock	Check and clean
	Lock mechanism faulty	Replace the part in question
Handle bar (Xpedition) will not hook on	Bar clasp (Xlevator) bent	Replace the part in question
System cannot be locked	Locking pin holes blocked	Check and clean
	Locking unit faulty	Replace the part in question
	Locking mechanism stiff	Lubricate or replace unit
System cannot be moved up or down	Lack of lubrication	Lubricate rails
	Gas spring and/or oil brake bent/faulty	Replace the part in question
	Other components bent/damaged	Contact fEX Engineering

Table 6: *Troubleshooting*

7.4 Maintenance by users

The parts listed below can be replaced by the user or operator with original spare parts. These can be ordered upon request.

- Bar clasp (Item no. FEX0140BH)
- Handle (Item no. FEX01400G)
- Strap (lock side) (Item no. FEX014GS1)
- Strap (latch side) (Item no. FEX014GZ1)
- Hygiene+ strap (lock side) (Item no. FEX014GS2)
- Hygiene+ strap (latch side) (Item no. FEX014GZ2)
- Strap holder (Item no. FEX0140GH)
- Left wheel mount (Item no. FEX014RAL)
- Right wheel mount (Item no. FEX014RAR)
- Edge protector (Item no. FEX014KS1)
- Sticker (Crush hazard) (Item no. FEX0140QG)
- Reflective strips (red) (Item no. FEX0140RR)
- Holding plate (Item no. FEX014HB1)

WARNING

Only replacement parts made by fEX Engineering GmbH may be used. Using other parts may be dangerous. **Repairs are prohibited.**

7.5 Changing the bar clasp

The bar clasp is used to secure the *Xpeditio* before attaching the straps.

Follow the steps below to replace the bar clasp:

1. Move the system to its highest position and lock into place.
2. Loosen the four M5x12 screws and the nuts behind them, and remove the damaged bar clasp.
3. Attach the new part using the new M5x12 screws/self-locking nuts with a tightening torque of 7 Nm.

7.6 Replacing the handle

Follow the steps below to replace the handle:

1. Move the system to its highest position and let it lock into place.
2. Loosen the four M5x12 screws on the back of the chair plate, and remove the damaged handle.
3. Apply a drop of medium-strength screw locking to the first (lowest) threads on **all** screws.
4. Attach the new part using the new M5x12 screws with a tightening torque of 7 Nm.

7.7 Replacing a strap

The straps and strap locks can be replaced separately. These are simply clipped into the strap anchors with a quick-release fastener.

Follow the steps below to replace a strap:

1. Open the strap lock.
2. Open the quick-release fastener (Fig. 8) and remove the damaged strap.
3. Clip the new strap into place, and lock the quick-release fastener into place.
4. Close the straps and lock diagonally over one another (Fig. 9).

Figure 8: Strap lock

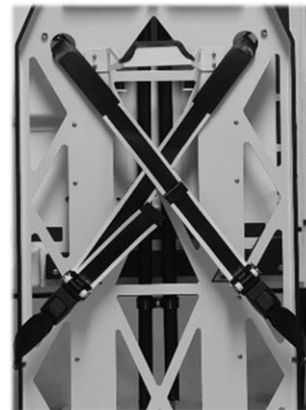
Figure 9: Correct strap position

7.8 Replacing the strap holder

The strap holder holds the free end of the strap so it does not get in the way during use.

Follow the steps below to replace the strap holder:

1. Open the strap lock.
2. Remove the strap on the latch side by opening the quick-release fastener (Fig. 8).
3. Push the strap holder over the quick-release fastener and pull the free end through.
4. Reconnect the strap and let the quick-release fastener locking into place.
5. Close the straps and lock diagonally over one another (Fig. 9).

7.9 Replacing the wheel mount(s)

The Xlevator has one left and one right wheel mount (Fig. 10) (1). Left and right are the corresponding sides when the Xlevator is viewed from the front as in Table 7: *Product overview* in Subsection 4.3 *Product overview*.

Follow the steps below:

1. Loosen the three M8 countersunk head screws (2) (Fig. 11) and remove the faulty wheel mount (1).
2. The right-angled inner edge marked in red (Fig. 10) is fitted flush. The opposite outer edge is rounded to fit over the weld seam. The right wheel mount is a mirror image of the left.
3. Attach the new wheel mount (1) using the three new M8x12 countersunk head screws (2) with a tightening torque of 8 Nm.

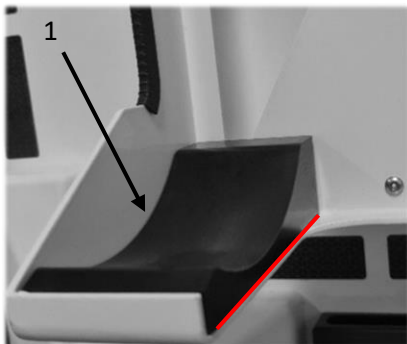
Figure 10: Wheel mounts

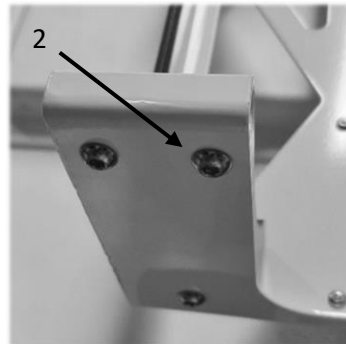
Figure 11: Screw connections for wheel mount

7.10 Replacing the edge protector

The Xlevator has an edge protector all the way round its outer edge to protect the Xlevator and the *Stryker Xpedition* from damage.

Follow the steps below to replace the edge protector:

1. Remove the damaged edge protector.
2. Start at one end of the chair plate.
3. Lift the free end and gradually push the edge protector over the plate (Fig. 12), until you reach the other end.
4. Cut off any excess with side cutters.

Figure 12: Replacing the edge protector

7.11 Replacing safety precautions and reflective strips

There are three “Caution: Crush Hazard” stickers (Fig. 13) and four reflective strips (two behind, one on the left and one on the right side).

To replace damaged stickers, follow the steps below:

1. Pull off the damaged stickers and remove any residual adhesive.
2. Position the new stickers in the same place as the old ones were.
3. Press on the new stickers firmly.

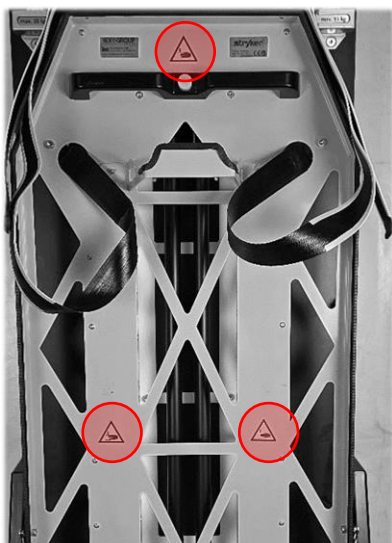
Figure 13: Safety precautions

7.12 Replacing the holding plates

There are two holding plates on the Xlevator to secure the Xpedition in vertical position. The holding plates are symmetrical and can be fitted on either side of the chair plate.

To replace damaged holding plates, follow the steps below:

1. Loosen the two M5x8 countersunk head screws and remove the damaged holding plate.
2. Attach the new holding plate with two new M5x8 countersunk head screws using a tightening torque of 5 Nm.

8 Dismantling, recycling, disposal

8.1 Dismantling

 **CAUTION**

The product must be cleaned and disinfected before being dismantled. Components that have been dismantled or damaged may have sharp edges.

The operator and his personnel may dismantle the product. It is the operator's responsibility to have qualified personnel carry this out. The product is dismantled in the reverse order of its assembly.

8.2 Recycling, disposal

When disposing of the product, always follow **current local** recommendations, and/or statutory provisions on environmental protection and risk relating to recycling or the disposal of devices at the end of their useful life.

8.3 Disposal through fEX Engineering GmbH

If the product has to be taken out of service due to damage or other reasons, the Xlevator (like all other fEX Engineering GmbH products) can be returned to the company for free disposal.

9 Risk management

9.1 Foreseeable misuse

The system must only be used for its intended purpose. The following foreseeable misuses are prohibited:

- Use without straps or with straps not tightly secured (securing)
- Transporting persons or other products/objects than the Stryker *Xpedition*
- Persons standing with their full body weight on the Xlevator
- Incorrect cleaning or disinfection
- Modification or repair measures by the user
- Adjusting, working on or opening the cylinders (gas spring/damper)
- Not inspecting or inspecting the product too infrequently
- Use of spare parts made by another manufacturer
- Cleaning with high-pressure
- Other uses contrary to the Instructions for use

Any improper use renders any liability or warranty from the manufacturer null and void.

9.2 Residual risk

Table 7 shows other factors to consider in order to minimise residual risk.

Residual risk	Measures to minimise residual risk
Objects become projectiles inside an emergency services vehicle	<ul style="list-style-type: none"> • Check that the <i>Xpedition</i> is stowed correctly and fully strapped in • Only use replacement parts from fEX Engineering GmbH
Crush hazard	<ul style="list-style-type: none"> • Do not put your fingers through the chair plate • Cleaning and disinfection should only be carried out in the locked position
Chair plate moves up/down uncontrollably/unintentionally	<ul style="list-style-type: none"> • Always ensure that the chair plate is fully locked
<i>Xpedition</i> falls off the Xlevator	<ul style="list-style-type: none"> • Use the bar clasp to secure the handle of the <i>Xpedition</i> • Check that the <i>Xpedition</i> is stowed correctly and fully strapped in
Potential transmission of pathogens	<ul style="list-style-type: none"> • Ensure the product is cleaned and disinfected properly according to the procedure in the Instructions for use. • Clean and disinfect the Xlevator after every possible contamination.
Sharp edges	<ul style="list-style-type: none"> • Make sure the product is handled and used correctly • Carry out weekly inspections • Remove damaged systems from operation immediately

Table 8: Measures to minimise residual risk

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8.3 Odstranjevanje prek podjetja fEX Engineering GmbH